

Category A Waste - UN3549

We now have a **60L WIVA™ container with a Solid lid**, for use **with two UN approved bulk clinical waste bags** which meets the requirement for UN3549 Category A Waste.

The standard 60L WIVA™ container with a Solid lid is available via your normal supply routes.

Instructions for use

- **Inner packaging** - UN approved bulk clinical waste bag, bag to be closed as per bag approval, to be placed inside the intermediate packaging.
- **Intermediate packaging** - UN approved bulk clinical waste bag, bag to be closed as per bag approval, to be placed in the outer packaging.
- **Outer packaging** - 60L WIVA™ container with a Solid lid.
- Attach UN3549 label to container overlaying UN3291 label – label available for download [here](#)
- When ready for transfer, consign appropriately with the following statement on the transport documentation “[Carriage agreed according to section 1.5.1 of ADR \(M317\)](#)”

Weight limit of 15 KG



For more information or a copy of the certificate please contact the customer support team via info.healthcare.uk@mauserpackaging.com or 01865 371 841.

Safety, Service and Support - Every Day, Affordably & Responsibly

www.daniels.co.uk



Download the App
for 24/7 support



MAUSER
Packaging Solutions

d.
DANIELS®
A Mauser Packaging
Solutions brand