

# SHARPSGUARD®

## Instructions for use



0.3L, 0.5L, 0.6L, 1L, 2.5L, 3.75L, 5L, 7L, 11.5L, 22L, 22Lxa, 22Lra, 24L, 24Lxa, theatre+ containers

### Assembly

The 0.3L, 0.5L, 0.6L & 1L containers are pre-assembled, endorse the label with your location and ward and the date it was started.

To assemble the 2.5L, 3.75L, 5L, 7L, 11.5L, 22L, 22Lxa, 22Lra, 24L, 24Lxa & Theatre+ containers place the lid on the container with the aperture at the front of the container by the label and press down firmly on each side, rotate the container 90° and press firmly again.

Check the container is assembled correctly before endorsing the label with your location and ward and the date it was assembled.



### Placement

Containers should be situated safely in accordance with your internal policies. Depending on the size of the containers there are POUDES® trays, brackets or mobile holders available if required.

### Safe Usage

- Between use, the door of the containers is to be put in the temporary closure.
- Avoid dropping sharp instruments into the bin from height, place them gently into the bin as indicated in the image.
- Container is to be filled to the fill line, never overfilled.
- Do not force sharps into the container.
- When container is filled to the fill line, the container is to be locked using the final closure and the label completed with the date it is locked.
- Once locked, do not attempt to re-open or re-use.
- If possible, use blade remover to detach blades from scalpel handle before disposal.
- Do not reach into the bin: Never attempt to retrieve or push down items inside the sharps bin.

### Handling

Use the carry handle when moving or handling the container. Care must be always taken when handling all sharps and clinical waste containers.

### Disposal

**Follow local regulations:** Dispose the bin in accordance with local laws for medical waste.

**Do not throw in regular trash:** Sharps bins must never be disposed of with household waste or recycling.

	0.3L	0.5L	0.6L	1L	2.5L	3.75L	5L	7L	11.5L	22L	22Lxa	22Lra	24L	24Lxa	theatre+
Allowable Usable Capacity (L)	0.24	0.4	0.49	0.84	2.21	2.98	3.98	5.80	9.27	17.86	16.98	17.26	20.06	20.06	32.0
Allowable Gross Mass (kg)	0.9	1.09	1.13	1.15	1.23	1.30	2.34	2.44	3.62	5.94	6.06	6.03	5.96	6.04	9.73
Maximum Filling Weight (kg)	0.17	1	1	1	1	1	2	2	3	5	5	5	5	5	8
Empty Weight of Container (g)	72	88	126	154	232	298	341	437	622	943	1036	1035	965	1048	1728

If you have questions or need further information, please contact your local Area Manager or our Customer Service Team on +44 1865 371841 or [info.healthcareUK@mauserpackaging.com](mailto:info.healthcareUK@mauserpackaging.com) and visit our website [www.daniels.co.uk](http://www.daniels.co.uk).



REDEFINING SUSTAINABILITY

[www.daniels.co.uk](http://www.daniels.co.uk)



[info.healthcareuk@mauserpackaging.com](mailto:info.healthcareuk@mauserpackaging.com)

Unless otherwise stated, the information contained herein is at the specific request of the user and has been made available by Daniels Healthcare.