

Information for users of our 30L Specimen container for Sharps transport

At Daniels Healthcare, safety and compliance are at the heart of everything we do. In collaboration with our Dangerous Goods Safety Advisor (DGSA), we have developed a new downloadable label to support a specific, alternative use of our 30L Specimen Container.

Why this label is needed

The 30L Transport Container is a robust, durable product originally designed for the transportation of diagnostic samples. However, following a detailed assessment with our DGSA, it has been confirmed that the container is suitable for a specific alternative use: **the transport of sharps containers secured in the temporary closure position.**

While the container itself is appropriate for this use, the original yellow label does not reflect the correct contents classification for this purpose. To maintain compliance with transportation regulations, a new label must be used.

This downloadable label ensures users can repurpose existing containers safely and compliantly, without the need to purchase new equipment.

How to use the label:

This label is for use on the 30L Specimen Container only.

It should only be used when the container is transporting sharps containers that are in the temporary closure position.

Apply the label correctly:

- Print the label as provided on page 2.
- Affix it directly over the original yellow label on the container.
- Ensure it is fully visible, securely attached, and not obscured.

Supporting safe and responsible reuse

This solution lets you extend the use of your current DANIELS® Transport Container in a compliant, clearly labelled way.

If you have questions or need further information, please contact your local Area Manager or our Customer Service Team on +44 (0)1865 371841 or info.healthcareuk@mauserpackaging.com and visit our website www.daniels.co.uk.

DANIELS® OVERPACK



To open: squeeze
together both sets of
catches and lift the lid.

To close: shut lid and
press.

RE-USEABLE
TRANSPORT
CONTAINER

UN3291
Clinical Waste
Unspecified, N.O.S.



30 Litre
Order code: DD8530P
LBDD8530P.V1

www.daniels.co.uk

DANIELS® OVERPACK



To open: squeeze
together both sets of
catches and lift the lid.

To close: shut lid and
press.

RE-USEABLE
TRANSPORT
CONTAINER

UN3291
Clinical Waste
Unspecified, N.O.S.



30 Litre
Order code: DD8530P
LBDD8530P.V1

www.daniels.co.uk

DANIELS® OVERPACK

To open: squeeze
together both sets of
catches and lift the lid.

To close: shut lid and
press.

RE-USEABLE
TRANSPORT
CONTAINER

UN3291
Clinical Waste
Unspecified, N.O.S.



30 Litre
Order code: **DD8530P**
LBDD8530P.V1

www.daniels.co.uk