

SHARPSGUARD[®] eco yellow 24

Assembly & Usage Instructions



- 1** Match the lid colour to the label colour. Line the lip at the front of the lid with the matching recess on base of the container.



- 2** Press the front and back until clicks are heard. Turn 90 degrees and repeat. Ensure the lid is securely attached by pressing around the lid rim. Fill in the Label.



- 3** Use the temporary closure between uses, gently press on the "Temp Close" on door to engage tab.



- 4** Flick the tab towards the front of the bin to disengage, then lift tab to open.



- 5** For final, permanent closure, press at back of door and slide thumbs forward following the arrows applying pressure to locking points.



- 6** Once secured, complete the "Locked by" section on the label.



If you have questions or need further information, please contact your local Area Manager or our Customer Service Team on +44(0)1865 371841 or Orders.UK@mauserpackaging.com and visit our website www.daniels.co.uk.



REDEFINING SUSTAINABILITY

www.daniels.co.uk



Orders.UK@mauserpackaging.com

Unless otherwise stated, the information contained herein is at the specific request of the user and has been made available by Daniels Healthcare.

V02.2025