## SHARPSGUARD® eco yellow 24

## Assembly & Usage Instructions



Match the lid colour to the label colour.
Line the lip at the front of the lid with the matching recess on base of the container.



Press both sides of the lid until clicks are heard. Turn 90 degrees and repeat. Ensure the lid is securely attached by pressing around the lid rim. Fill in the label.



Use the temporary closure between uses, gently press on the "Temp Close" on door to engage tab.



Flick the tab towards the front of the bin to disengage, then lift tab to open.



For final, permanent closure, press at back of door and slide thumbs forward following the arrows applying pressure to locking points.





Once secured, complete the "Locked by" section on the label.



If you have questions or need further information, please contact your local Area Manager or our Customer Service Team on +44(0)1865 371841 or Orders.UK@mauserpackaging.com and visit our website www.daniels.co.uk.



**REDEFINING SUSTAINABILITY** 



Unless otherwise stated, the information contained herein is at the specific request of the user and has been made available by Daniels Healthcare.

