

SHARPSGUARD[®] *eco pharmi 24 xa*

Assembly & Usage Instructions



- 1** Match the lid colour to the label colour. Line the lip at the front of the lid with the matching recess on base of the container.



- 2** Press the front and back until clicks are heard. Turn 90 degrees and repeat. Ensure the lid is securely attached by pressing around the lid rim. Fill in the Label.



- 3** Use the temporary closure between uses, press on "Temp Close" on either side of the door to engage.



- 4** Lift door to open for use.



- 5** For final, permanent closure, apply pressure to "Press to lock" until a click is heard.



- 6** Once secured, complete the "Locked by" section on the label.



If you have questions or need further information, please contact your local Area Manager or our Customer Service Team on +44(0)1865 371841 or Orders.UK@mauserpackaging.com and visit our website www.daniels.co.uk.



REDEFINING SUSTAINABILITY

www.daniels.co.uk



Orders.UK@mauserpackaging.com

Unless otherwise stated, the information contained herein is at the specific request of the user and has been made available by Daniels Healthcare.

V02.2025