

SHARPSGUARD[®] *eco pharmi 24 xa*

Assembly & Usage Instructions



- 1** Match the lid colour to the label colour. Line the lip at the front of the lid with the matching recess on base of the container.



- 2** Press both sides of the lid until clicks are heard. Turn 90 degrees and repeat. Ensure the lid is securely attached by pressing around the lid rim. Fill in the label.



- 3** Use the temporary closure between uses, press on "Temp Close" on either side of the door to engage.



- 4** Lift door to open for use.



- 5** For final, permanent closure, apply pressure to "Press to lock" until a click is heard.



- 6** Once secured, complete the "Locked by" section on the label.



If you have questions or need further information, please contact your local Area Manager or our Customer Service Team on +44(0)1865 371841 or Orders.UK@mauserpackaging.com and visit our website www.daniels.co.uk.



REDEFINING SUSTAINABILITY

www.daniels.co.uk



Orders.UK@mauserpackaging.com

Unless otherwise stated, the information contained herein is at the specific request of the user and has been made available by Daniels Healthcare.

V02.2025