

WIVA™ 60L Solid lid


Ⓜ 3H2W/X15/S/26/GB/7425 REC

DANGER

**DESTROY BY
INCINERATION**

OR OTHER SUITABLY
AUTHORISED PROCESS

**HAZARDOUS WASTE,
INFECTIOUS CLINICAL WASTE**

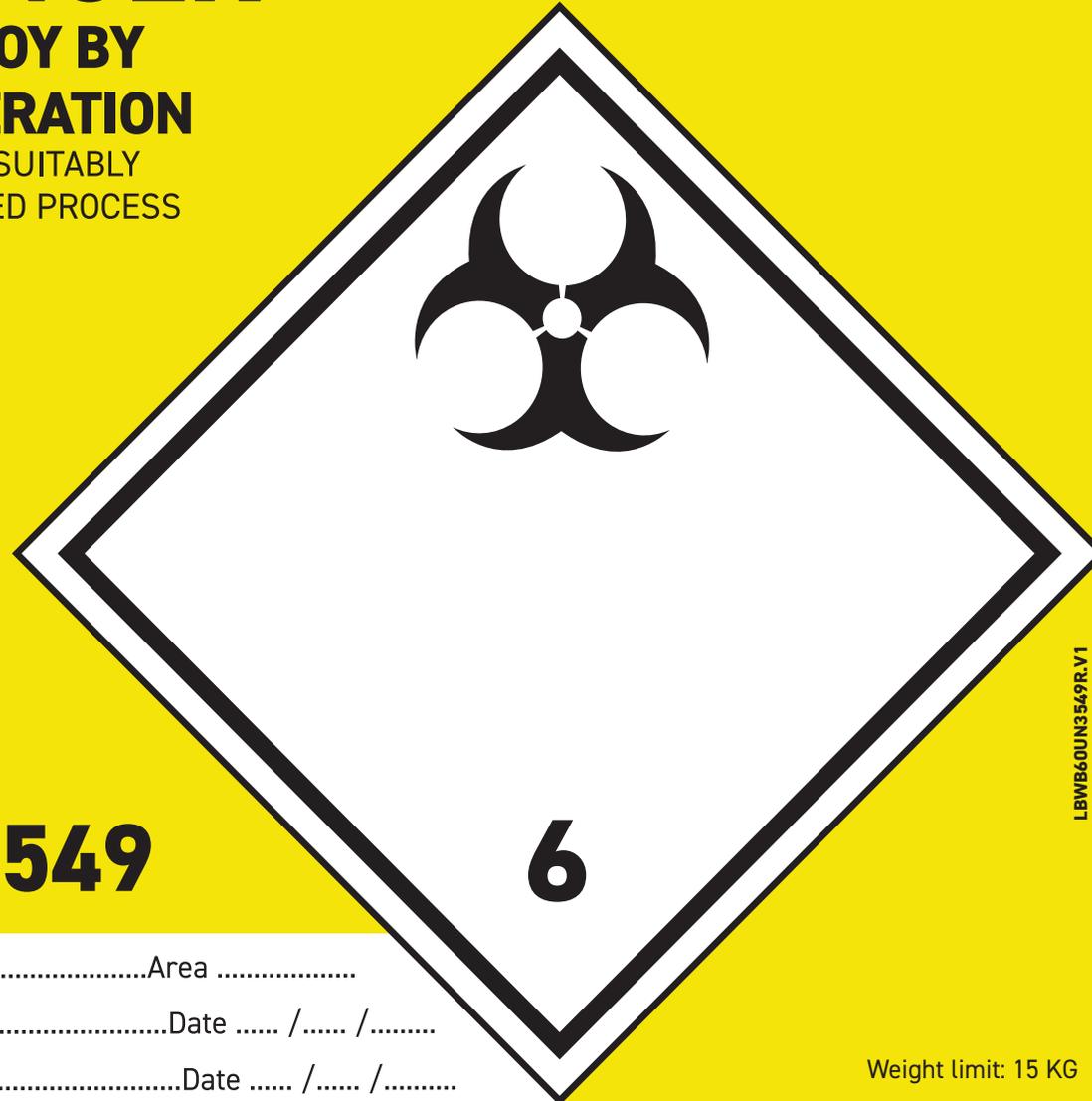

UN3549

LBWB60UN3549R.V1

Hospital.....Area

Started by.....Date / /

Closed by.....Date / /

Weight limit: 15 KG

www.daniels.co.uk



Instructions for use

· Inner packaging - UN approved bulk clinical waste bag, bag to be closed as per bag approval, to be placed inside the intermediate packaging.

· Intermediate packaging - UN approved bulk clinical waste bag, bag to be closed as per bag approval, to be placed in the outer packaging.

· Outer packaging - 60L WIVA™ container with a Solid lid.

· Attach UN3549 label to container overlaying UN3291 label – label available for download on our website

· When ready to be transferred waste is to be consigned appropriately and the following statement to be included on the transport documentation "Carriage agreed according to section 1.5.1 of ADR (M317)"

WIVA™ 60L Solid Lid